

# Matrix® FLIPP LOXX™

## SET-SCREW ADJUSTMENT INSTRUCTIONS

### IMPORTANT!

If the latch mechanism on the Flipp-Loxx headrest assembly does not engage properly in the upright/locked position, a **set-screw adjustment** will be required. Refer to the set-screw adjustment instructions below.

### LATCH MECHANISM/SET SCREW ADJUSTMENT INSTRUCTIONS: (see images below)

1. Flip the headrest to expose the **set screw (H)**.
- 2a. If there is too much play in the latch mechanism (too loose):
  - rotate the set-screw **counterclockwise\***
- 2b. If the latch mechanism will not engage:
  - rotate the set-screw **clockwise\***
3. Return the headrest to the upright position and check to see if the latch mechanism is fully engaged/locked.
4. Repeat set-screw adjustment as necessary, until proper set-up is achieved  
(\*adjust using 2.5mm hex key provided)

