

## Invacare Distributors:

Belgium & Luxembourg:  
Invacare nv Autobaan 22  
B-8210 Loppem  
Tel: (32) (0)50 83 10 10  
Fax: (32) (0)50 83 10 11  
belgium@invacare.com  
www.invacare.be

España:  
Invacare SA  
c/Areny s/n, Polígon Industrial  
de Celrà  
E-17460 Celrà (Girona)  
Tel: (34) (0)972 49 32 00  
Fax: (34) (0)972 49 32 20  
contactsp@invacare.com  
www.invacare.es

Danmark:  
Invacare A/S Sdr. Ringvej 37  
DK-2605 Brøndby  
Tel: (45) (0)36 90 00 00  
Fax: (45) (0)36 90 00 01  
denmark@invacare.com  
www.invacare.dk

France:  
Invacare Poirier SAS  
Route de St Roch  
F-37230 Fondettes  
Tel: (33) (0)2 47 62 64 66  
Fax: (33) (0)2 47 42 12 24  
contactfr@invacare.com; www.invacare.fr

Deutschland:  
Invacare GmbH  
Alemannenstraße 10  
D-88316 Isny  
Tel: (49) (0)7562 700 00  
Fax: (49) (0)7562 700 66  
kontakt@invacare.com; www.invacare.de

Ireland:  
Invacare Ireland Ltd,  
Unit 5 Seatown Business Campus  
Seatown Road, Swords,  
County Dublin  
Tel : (353) 1 810 7084  
Fax: (353) 1 810 7085  
ireland@invacare.com; www.invacare.ie

Eastern Europe, Middle East & CIS:  
Invacare EU Export Kleiststraße 49  
D-32457 Porta Westfalica /  
Germany  
Tel: (49) (0)57 31 754 540  
Fax: (49) (0)57 31 754 541  
webinfo-eu-export@invacare.com  
www.invacare-eu-export.com

Italia:  
Invacare Mecc San s.r.l.,  
Via dei Pini 62,  
I-36016 Thiene (VI)  
Tel: (39) 0445 38 00 59  
Fax: (39) 0445 38 00 34  
italia@invacare.com; www.invacare.it

Nederland:  
Invacare BV Galvanistraat 14-3  
NL-6716 AE Ede  
Tel: (31) (0)318 695 757  
Fax: (31) (0)318 695 758  
nederland@invacare.com  
www.invacare.nl

Suomi:  
Camp Mobility  
Patamäenkatu 5,  
33900 Tampere  
Puhelin 09-35076310  
info@campmobility.fi  
www.campmobility.fi

Norge:  
Invacare AS  
Grensesvingen 9, Postboks 6230,  
Etterstad  
N-0603 Oslo  
Tel: (47) (0)22 57 95 00  
Fax: (47) (0)22 57 95 01  
norway@invacare.com  
www.invacare.no

Sverige:  
Invacare AB Fagerstagatan 9  
S-163 53 Spånga  
Tel: (46) (0)8 761 70 90  
Fax: (46) (0)8 761 81 08  
sweden@invacare.com  
www.invacare.se

Österreich:  
Invacare Austria GmbH Herzog Odilostrasse  
101  
A-5310 Mondsee  
Tel: (43) 6232 5535 0  
Fax: (43) 6232 5535 4  
info-austria@invacare.com  
www.invacare.at


Schweiz / Suisse / Svizzera:  
Invacare AG Benkenstrasse 260  
CH-4108 Witterswil  
Tel: (41) (0)61 487 70 80  
Fax: (41) (0)61 487 70 81  
switzerland@invacare.com  
www.invacare.ch

Portugal:  
Invacare Lda  
Rua Estrada Velha, 949  
P-4465-784 Leça do Balio  
Tel: (351) (0)225 1059 46/47  
Fax: (351) (0)225 1057 39  
portugal@invacare.com  
www.invacare.pt

United Kingdom:  
Invacare Limited  
Pencoed Technology Park,  
Pencoed  
Bridgend CF35 5AQ  
Tel: (44) (0) 1656 776 222  
Fax: (44) (0) 1656 776 220  
uk@invacare.com; www.invacare.co.uk

USA:  
Invacare USA  
Worldwide Headquarters  
One Invacare Way  
Elyria, Ohio,  
USA 44036-2125  
Tel: (1) 800 333 6900  
Fax: (1) 877 619 7996  
www.invacare.com

Canada:  
Invacare Canada  
570 Matheson Blvd. E. Unit 8  
Mississauga, Ontario  
Canada L4Z 4G4  
Tel: (800) 668 5324  
Fax: (905) 890 5244  
www.invacare.ca

 Motion Concepts LP  
84 Citation Drive, Unit 1  
Concord, Ontario L4K 3C1 Canada

**EC REP**

Advena Ltd.  
Tower Business Center,  
2nd Flr., Tower Street,  
Swatar, BKR 4013 Malta


**Motion Concepts USA**  
700 Ensminger Rd. Suite 112  
Tonawanda, NY 14150  
Toll Free Tel: 1.888.433.6818  
Toll Free Fax: 1.888.433.6834

**Motion Concepts Canada**  
Toll Free Tel: 1.866.748.7943  
Tel: 905.695.0134  
Fax: 905.695.0138

  
**Concepts  
Motion**  
www.motionconcepts.com



# matrix® LOXX™ LOXX HEADREST MOUNTING HARDWARE

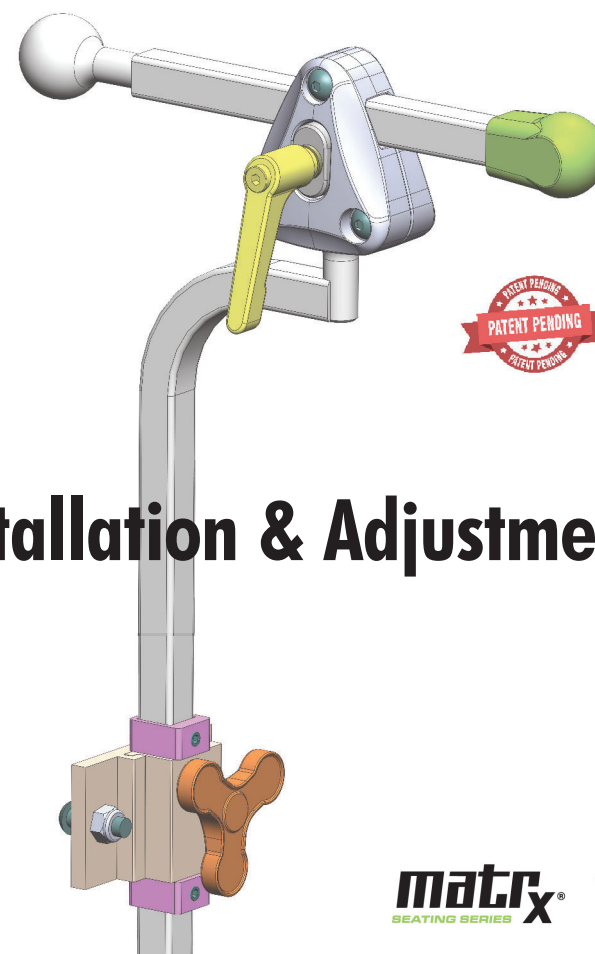


## Installation & Adjustment


**matrix®**  
SEATING SERIES

**INVACARE**  
Yes, you can.™

# MATRIX LOXX™ MOUNTING HARDWARE

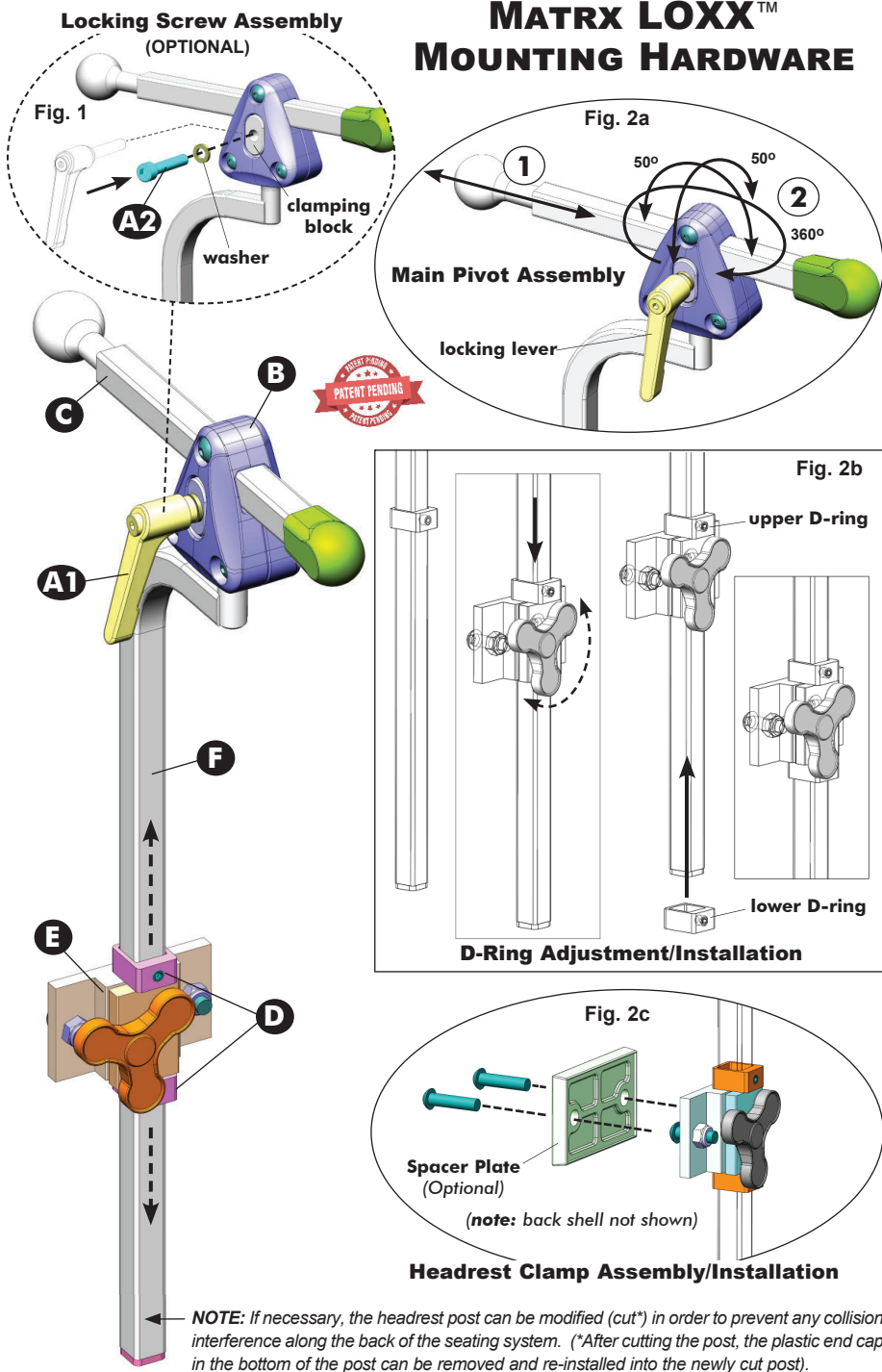

## HEADREST PADS: (SOLD SEPARATELY)

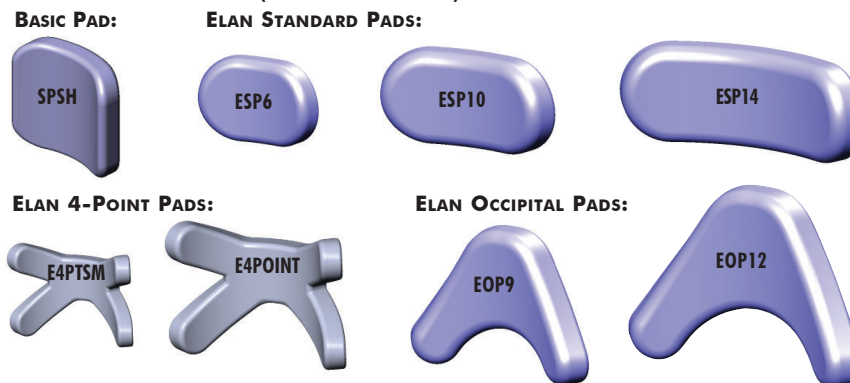

### Headrest Pad Installation

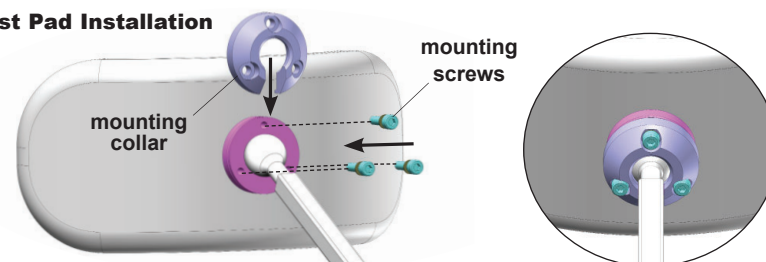


Fig. 3 Note: mounting hardware/collar included with headrest pads

## MATRIX LOXX™ (HEADREST) INSTALLATION/ADJUSTMENT:

**OPTIONAL LOCKING SCREW:** If preferred, the locking lever (A1) may be replaced with the 30mm locking screw (A2) provided. Remove/unscrew the lever from the upper clamp housing and retain the original washer. Being careful to keep the clamping blocks in position inside the housing, use the 5mm hex key (provided) to install the replacement locking screw (with washer) -see Fig. 1.

1. Secure mounting clamp assembly (E) onto back shell (or headrest adapter plate) using the hardware provided (see Fig. 2c). If additional clearance is required, install the spacer plate with longer screws provided. (*note: mounting clamp hardware is installed from the inside of the back shell*)
2. Install headrest pad (*sold separately*) onto horizontal bar (C) hardware via the mounting hardware provided with headrest pad -see Fig. 3 above.
3. Loosen/remove lower D-Ring (D) from headrest post (F) using 2.5mm hex key.
4. Slide the headrest post (F) into the mounting clamp assembly (E). Loosen upper D-Ring (D) and adjust the headrest pad/post to the desired height (see Fig. 2b). Tighten the mounting clamp (knob) to secure the headrest post in place. Slide the upper D-Ring so that it rests flush on top of the mounting clamp, then secure it into place via set-screw.
5. To Adjust Depth/Angle of the Headrest Pad: Loosen the locking lever (A1) or locking screw (A2) (*approx. 1/2 - 3/4 turn*) to unlock the upper clamp housing (B). Slide the horizontal bar (C) to the desired depth inside the upper clamp (see Fig. 2a (1)); Adjust the headrest angle by pivoting/rotating the upper clamp to achieve the desired headrest position (see Fig. 2a (2)).
6. Once the final headrest position is set, re-tighten the upper clamp (via locking lever/locking screw) and verify that ALL Matrix LOXX™/Headrest hardware is fully tightened.
7. To fully lock the headrest into place (and prevent slipping), reinstall/secure the Lower D-Ring onto the headrest post so that it rests flush with the bottom of the mounting clamp assembly (see Fig.2b).