

CUPHOLDER

MOB1024GB & MOB1091XL

WHAT'S INCLUDED

- 1x Cupholder with Telescoping Base
- 1x Clamp Piece



To see all the FAQ's,
visit vhealth.link/4gq

vive

QUICK START GUIDE

Step 1:

Unthread the plastic "twist knob" on your (B) Clamp Piece so that it is barely thread onto the thread of the Clamp.



Step 2:

Insert the Ball end of the Ball Piece into the hole in the threaded part of the clamp. (Note this piece is pre-attached to the cupholder, It will be easier to slide this piece off the cupholder part to complete the next step).



NOTE: This will take some force to push this part in.

Step 3:

Tighten the "twist knob" down.

NOTE: If you want to use a maximum size full cup (40oz cup) you must tighten this down all the way and really use some twisting force. If you do get it tight enough there you will be impressed with the grip strength.



QUICK START GUIDE

vive

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
1-800-487-3808
www.vivehealth.com

Step 4:

Insert the cupholder piece and the clamp piece together and slide it down slightly into the slots to lock it into place.



Step 5:

Your cupholder is finished and you may use the turning screw on top of the clamp to clamp down on any 1.75" wide or smaller fixture.



PRO TIP: For heavier cups if you angle the cuphold at an upward angle (just slightly) from the clamp, it will hold heavy objects better. To angle the cupholder in any position just loosen the twist knob slightly, reposition the cupholder on the ball joint and then retighten.