

# ALL TERRAIN KNEE WALKER

MOB1019

**vive**®  
MOBILITY



To see all FAQs in  
one place, visit  
[vhealth.link/4e132](https://vhealth.link/4e132)

# ALL TERRAIN KNEE WALKER

## WHAT'S INCLUDED



- Front Frame (A)
- Rear Frame (B)
- Knee Pad (C)
- Steering Column/Handle Bars (attached by wire to the Front Frame) (D)
- Storage Bag (E)
- 3x Screw Knobs (F)


## ASSEMBLY INSTRUCTIONS

No Tools Required!





Want to watch us do this?  
Check out [vhealth.link/8aa0b](https://vhealth.link/8aa0b)  
for an easy instructional video.

1. Pull the steering column of the Front Frame (A) to the vertical position. When in place, the silver pin running through the tension lever assembly will lock the two sections together.
2. Take the Front Frame (A) and insert the 2 male ends into the female ends on the Rear Frame (B). Push down the locking pins and push the frame pieces together until the locking pins engage and are locked in place.

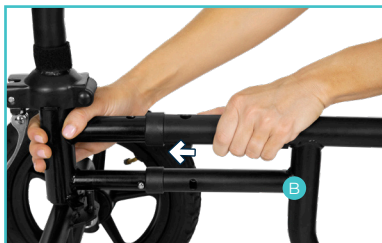


3. Insert a Screw Knob (F) into the top of the frame above the locking pins and twist until fully tightened to secure the scooter wheelbase.

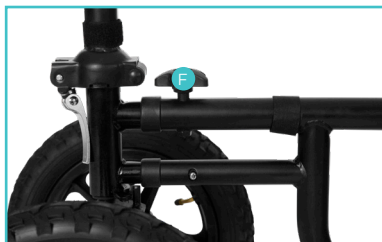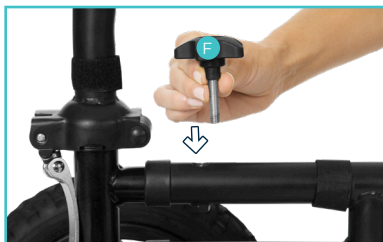

4. Pull the silver tension lever assembly up until the lever and the spring bolt are vertical and parallel to the steering column socket.
5. Pull just the flat silver lever back down to apply tension to the folding hinge and stabilize the front end of the scooter. If the silver lever does not apply enough tension, lift the lever back up, tighten the nut at the bottom of the spring bolt by half a turn, and retry pulling the lever back down. Repeat as necessary to ensure the lever provides enough tension.
6. Attach the handle bars to the steering column. Make sure that the brake line is not remove twisted around the bar.



- Adjust the Handle Bars (D) to the desired height making sure that the holes for inserting the Screw Knob (F) line up. Secure the handle bars with the Screw Knob (F). Twist clockwise until fully tightened.

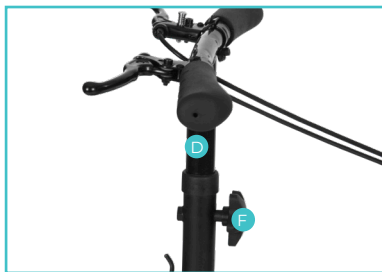

- Insert the Knee Pad (C) post into receptacle on the Rear Frame (B). Lock the pad in place by inserting a Screw Knob (F) into the desired adjustable height hole making sure that the holes for inserting the locking knob line up. Twist the knob clockwise until fully tightened.



9. Take the brake line and secure it inside the two fabric straps you will find on the top bar of the frame as well as the steering column.



10. To attach the Storage Bag (E), locate the two hooks on the Front Frame (A) and the sleeve on the back of the bag. Insert the sleeve over the hooks. Attach the hook and loop fastener on the bottom of the bag to the steering column.



## OPERATING INSTRUCTIONS

### General Information

- Position your injured knee and leg firmly and securely in the middle of the knee pad, using the entire length of the pad to support your leg.
- While your knee is on the walker, stand as straight as possible. Adjust the height of the handlebars and the knee pad for comfort.

## To Lock the Brakes



To see a video demonstration, follow the link here: [vhealth.link/8ddf7](https://vhealth.link/8ddf7)

- Pull the hand brake lever in towards you to engage the brake.
- With the lever pulled in and the brake engaged, press the silver locking button down with your thumb.
- Hold the button down while releasing your grip on the brake lever. The pin will hold the tension on the brake cable and lock the brakes in place.

## To Unlock the Brakes

- Pull on the brake lever until the silver locking button pops up, which release the tension on the brake cable and unlocks the brakes.




## Folding the Steering Column

- Lift up the silver lever to release the tension in the folding hinge on the Front Frame (A) .
- Rotate the silver lever and spring bolt assembly down.
- Pull the spring bolt assembly to the left to unlock the folding hinge, and push the steering column back to fold the handlebars down.

### NOTE:

Removing the Knee Pad is optional. The handlebars can be tucked under the Knee Pad if desired.

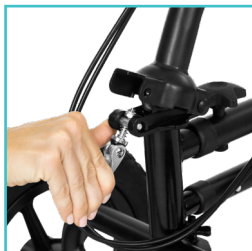



## SPECIFICATIONS

- Length: 33.5"
- Width: 18"
- Handlebar Height: 37.5" - 45.25"
- Wheels Diameter: 12"
- Weight: 25.8 lbs
- Knee Pad Length: 13.5"
- Knee Pad Width: 7"
- Knee Pad Adjustable Height: 19" - 24"
- Height with handle bar folded down: 19.5"
- Maximum User Weight: 350lbs

## WARNINGS

- Always consult with your physician / therapist before use to determine proper adjustment and use. As with any new physical activity, common sense and practice are critical for the safe operation of the Knee Walker.
- Be sure you are on a level surface with the brake \ lock engaged when mounting / dismounting the Knee Walker.
- The hand brake is designed to maintain the Knee Walker in a stopped position. The hand brake is not intended to be used for stopping the Knee Walker while in motion. Using the hand brake while in motion may cause an abrupt stop and a loss of stability in the device.
- Use caution when moving between floors / surfaces with different textures (ex. moving from carpet to tile, or a concrete sidewalk to an asphalt street)

- Do not hang anything from the frame of the Knee Walker, as it may tilt the device off-balance.
- Do not adjust the screws / bolts on the device unless otherwise specified in the instructions.
- Failure to properly secure the components will cause the Knee Walker to be unstable and difficult to control.

## NOTES

---

---

---

---

---

---

---

---

---

---



## HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at [vhealth.link/4e132](https://vhealth.link/4e132) for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, email, or on our website

[sales@aracent.com](mailto:sales@aracent.com)  
888-550-7750  
[www.aracent.com](http://www.aracent.com)

Distributed by



8955 Fontana Del Sol Way  
Naples, FL 34109  
Made in China