



BARIATRIC ROLLATOR

Owner's Manual
MOB1040



To see all of the FAQs in one place,
visit vhealth.link/1yy

WHAT'S INCLUDED

- 1x Bariatric Rollator
- 1x Padded Backrest Bar
- 2x Bolts
- 2x Curved Washers
- 2x Star Knobs
- 1x Under Seat Bag

ASSEMBLING THE BARIATRIC ROLLATOR

CAUTION: Do not attempt to adjust or operate the Vive Bariatric Rollator without carefully reading all instructions in this manual.

1. Unfold the frame and slide the padded backrest into the frame brackets. Ensure the snap buttons protrude through the holes in the brackets.



2. Remove the star knobs, the curved washers and the hex bolts from the packaging and insert the handle tube into the frame at the desired height, lining up the holes appropriately. Slide the bolt through the holes from the inside and tighten the star knob on the outside of the frame until the assembly is secure.



3. Snap secure the bag underneath the seat.



SAFETY PRECAUTIONS REGARDING ASSEMBLY

Perform the following safety checks and resolve any issues before using the Rollator.

1. Ensure the Bariatric Rollator is fully unfolded/opened, that the seat is down, and that the folding bar underneath the seat is locked in the down position.
2. Ensure that both knobs are fully tightened and at equal heights on each side of the rollator. Test the rollator for stability before applying full weight to the hand grips.
3. Test the loop locks before operating the rollator. When the loop lock braking system is engaged (see illustrations for locking and releasing locks below), the locking bar on the rear wheels should prevent the wheels from rolling. If the wheels roll while the brakes are locked, please contact your dealer for adjustment. When the locks are released, the wheels should roll freely.
4. Always lock the loop locks before sitting and do not use the seat when the unit is on an incline or uneven ground.
5. Replace the tires when they become significantly worn or if they become damaged.
6. Do not push a person seated on the rollator. The rollator is a walking aid only and not to be used as a transportation device.
7. Do not exceed the weight capacity of 500 lbs / 226 kg.

OPERATING THE BRAKES

Always engage and disengage the wheel locks on both sides simultaneously. To operate the Brakes:

1. To Slow Down or Come to a Temporary Stop - Pull up slowly on the brakes with four fingers until the unit slows down or comes to a stop.
2. To Temporarily Lock the Wheels in Position - Press down on both hand brakes with the palms until you hear a "click" that indicates that the wheels are locked.

3. To Release the Wheel Locks - Pull up on both loop locks.



MAINTENANCE AND CARE INSTRUCTIONS

Vive recommends periodic visual inspection of this product by the user or the user's caregiver(s) to ensure that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit vivehealth.com for more information.

WARNINGS

- Always lock the loop locks before sitting and do not use the seat when the unit is on an incline or uneven ground.
- Replace the tires when they become significantly worn or if they become damaged.
- Do not push a person seated on the rollator. The rollator is a walking aid only and not to be used as a transportation device.
- Do not exceed the weight capacity of 500 lbs / 226 kg.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at
vhealth.link/1yy for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



sales@aracent.com



1-888-550-7750



aracent.com

Distributed by

vive
health

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China