



EVERGLIDE ROLLATOR

Owner's Manual
MOB1057




To see all FAQs in one place
visit **vhealth.link/6vu**

WHAT'S INCLUDED

- Ⓐ Frame
- Ⓑ Front Wheels
- Ⓒ Back Wheels and Arms
- Ⓓ Bolt
- Ⓔ Washer
- Ⓕ Lock Knob
- Ⓖ Cane Holder
- Ⓗ Backrest
- Ⓘ Bag



1. Place the Front Wheels (B) onto the Frame (A) by placing them into the front holes of the frame. Line up the hole on the wheel post with the hole that was made in the frame so that you may lock the screw into place.

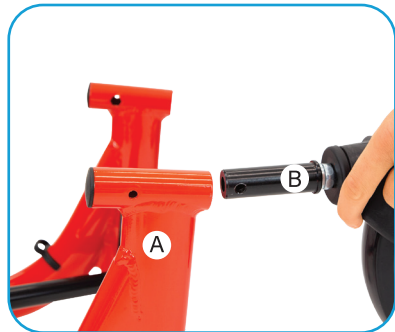

2. Place the Screw (D) so that the hex hole on the frame and the hex head of the screw are able to fit together.



On the other end of the screw, place the Washer (E).



Then lock it all together with the Locking Knob (F) (Repeat this step for the second front wheel).



3. Place the (C) Back Legs (Which are attached to the arms of the rollator via the brake line) to the rear of the Frame. Push in the locking pin on the back leg in so that you may insert them into the frame and make sure the pin depresses in the hole so they are locked in place. **NOTE:** Make sure when placing the back legs that the wheels are on the OUTSIDE of the frame and not on the inside where the user will be walking. Repeat this for the other back wheel.



3. Place the Arms that are attached to the back wheels into the top of the frame. Make sure the brake lines do not cross each other and run them along the outside of the frame. To insert the arms push in the locking pin on the bottom of the arm in so that you may insert them into the frame and make sure the pin depresses in the hole so they are locked in place. Choose the height that is most comfortable for you. Place the brake line inside the black plastic clips on the frame that are designed to hold the line.



5. Place the (G) Cane Holder onto the black knob on the side of the frame.



6. Add the Bag (I) to the front on the rollator using the loops on the bag to place them on the black pegs that protrude outward from the seat.



7. Insert the Backrest (H) onto the black slots located on the arms of the Rollator. These Slots make the backrest height adjustable by loosening the knobs on each of the sides.



Make sure this is locked down very well before using the seat.



Locking Brakes

Locking the brakes is an important feature of this rollator, especially when using the seat portion of the Rollator. To lock the brakes press DOWN on the brake lever on both sides until it clicks and holds in the downward position.



SPECIFICATIONS

Max User Weight: 300 LBS
Overall Width: 24.25"
Overall Length: 27"
Armrest Lowest Height: 33"
Armrest Highest Height: 38"
Seat Width: 17.5"
Seat Depth: 10"

WARNINGS

- Do not use strong corrosive cleaner to clean the product's surface.
- Product should not be left outside in rainy conditions.
- Product may rust if regularly exposed to moisture.
- Before sitting down on the seat, please lock both brakes to prevent the rollator from sliding.
- Do not hang any heavy weighted items on either of the armrests as it will cause the rollator to be unstable.
- Please fold the rollator before storage.
- This item is meant to be a walking aid. This means that it is not built to Support the full weight of one's body, it is only meant to assist you while walking. If a large amount of force is placed on this item, even when brakes are in locked position, this can cause the item to move out of its position.

HAVE MORE QUESTIONS?

Check out our list of Frequently Asked Questions at **vhealth.link/6vu** for helpful answers.



And if that doesn't answer your question, our customer service team would love to help! Feel free to connect with them by phone, e-mail, or chat on our website.



sales@aracent.com



1-888-550-7750



aracent.com

Distributed by

**vive
health**

8955 Fontana Del Sol Way
Naples, FL 34109
Made in China